NUK®

JOLIE

Manual Breast Pump
Manüel Gögüs Pompası

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



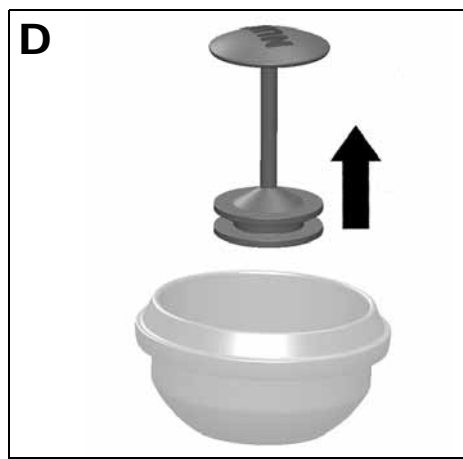
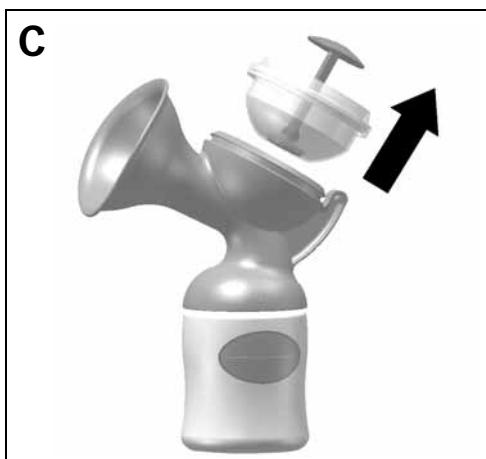
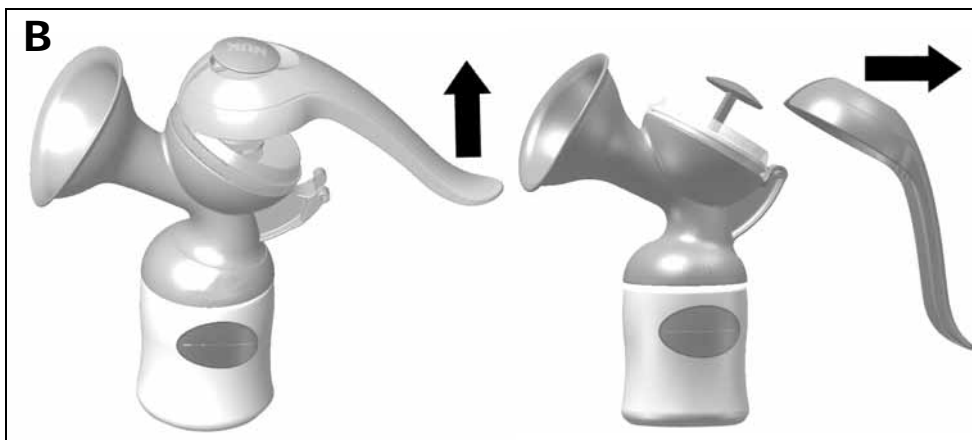
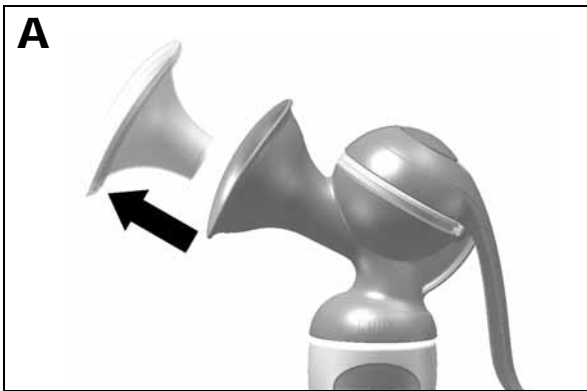
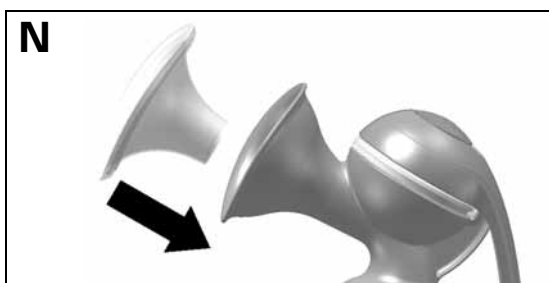
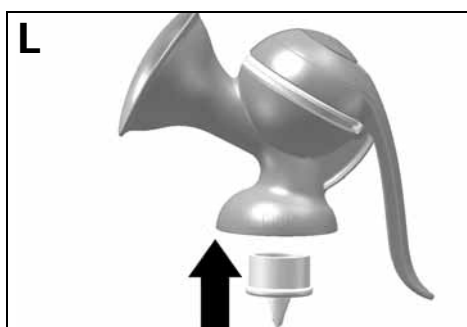
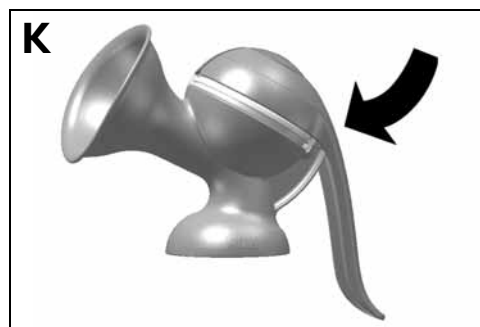
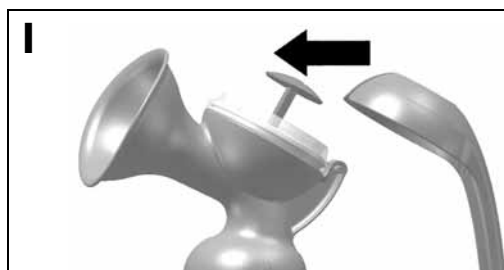
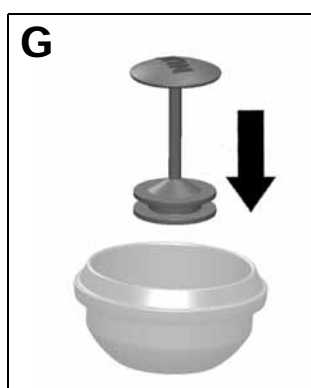
O

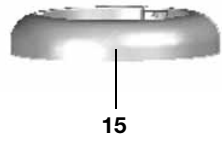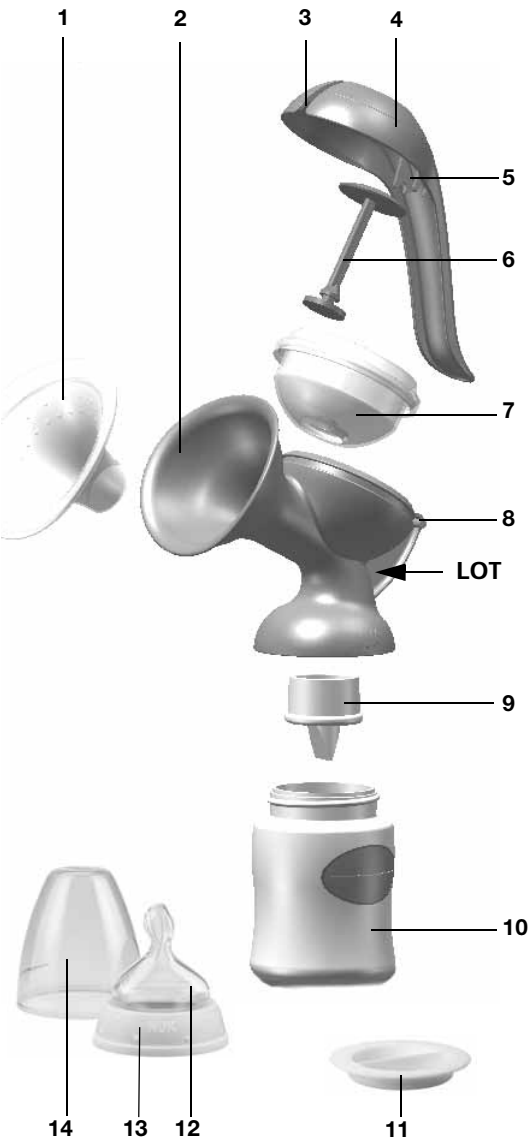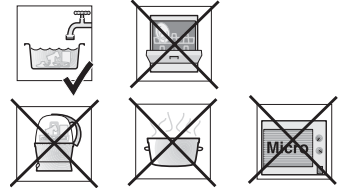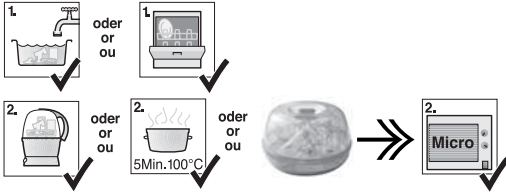Contents

1. Important information before using for the first time	4
2. Pump description	4
2.1 Intended use	4
2.2 Description of function	4
2.3 Contra-indications	4
2.4 Side effects	4
3. Safety information.	5
3.1 General information	5
3.2 Hygiene	5
3.3 NUK Breast Milk Container	5
4. Key parts (Figure O)	6
5. Taking the pump apart (Figures A to F).	7
6. Cleaning and sterilising	7
6.1 Cleaning	7
6.2 Sterilising	7
7. Reassembling the pump (Figures G to N).	8
8. Using the breast pump	8
9. Tips for successful expressing	9
9.1 Frequency of expressing	9
9.2 Method	9
9.3 Quantity of milk	9
9.4 Let-down reflex	9
10. Hygiene tips.	9
11. Storing milk	9
11.1 NUK Breast Milk Container	9
11.2 Important information for handling breast milk	10
12. Help with problems.	11
13. Technical data	11
14. Disposal	12
15. Warranty	12

Dear Mother,

Breast milk is the best food for a baby during the first six months of its life. To enable you and your baby to enjoy every moment of breastfeeding, NUK has developed a range of practical and comfortable breastfeeding aids for the systematic support of breastfeeding.

The NUK JOLIE Breast Pump is designed to help you and your baby whenever natural breastfeeding is not possible.

Before using your NUK JOLIE Breast Pump for the first time, please allow a few minutes of your time to carefully read these instructions for use.

1. Important information before using for the first time

Before using the NUK JOLIE Breast Pump for the first time, clean and sterilise all parts except part 15 (see “6. Cleaning and sterilising“ on page 7).

2. Pump description

2.1 Intended use

The NUK JOLIE Breast Pump is an aid to expressing and collecting breast milk:

- in the workplace, when baby is away, at home or when travelling
- to maintain milk production during a break in breastfeeding
- to relieve engorgement
- to increase milk production

Once expressed, breast milk can be collected, stored, frozen, warmed and fed to your baby using the NUK Breast Milk Container.

2.2 Description of function

The NUK JOLIE Breast Pump is a manual suction pump and offers the following benefits:

- simple to use and needs no power supply
- breast pump comes apart for easy cleaning
- silicone cushion for gentle massaging effect and softer feel

- imitates your baby's natural suckling action
- collects expressed breast milk
- supplied with stand for safe, convenient storage

2.3 Contra-indications

If you suffer from certain breast ailments (inflammation, open wounds), please consult your doctor, midwife or breastfeeding counsellor before using the NUK JOLIE Breast Pump.

2.4 Side effects

If at any time you experience an unexpected reaction, please consult your midwife, health visitor, doctor or pharmacist.

3. Safety information

Breastfeeding is best for your baby.

For your own safety please observe the following:

3.1 General information

- Before using the pump, please read these instructions for use carefully. These instructions are part of the unit and must be kept to hand at all times.
- Use the NUK JOLIE Breast Pump and accessories for their intended purpose only (see “2.1 Intended use”)
- To avoid the risk of choking, keep small parts that may be swallowed out of the reach of children.
- We recommend removing any fatty or oily creams from your breast before using the pump, as these may damage the surface of the breast shield.
- To guarantee that the pump will function correctly, only use original NUK accessories.
- Please note that any claim under warranty or liability will be void if accessories are used which are not recommended in these operating instructions, or if any parts other than original spare parts are used for repairs. This also applies to repairs carried out by unqualified persons.
- If in the event of any malfunctions, please contact manufacturer.
- If the breast milk is for a premature baby, it is essential to follow your doctor's instructions.

3.2 Hygiene

- To ensure the healthy development of your child, when handling breast milk, hygiene is of the utmost importance.

- Remove all body jewellery (e.g. piercings) before using the breast pump.
- To prevent infection or bacterial contamination (microbial growth) and to prevent damage to the pump, please read and pay attention to “6. Cleaning and sterilising” on page 7.
- **Only a breast pump which has been thoroughly cleaned will achieve full suction strength.**

3.3 NUK Breast Milk Container

3.3.1 Preparation

- Do not microwave bottles filled with feed or water (risk of explosion/material damage).
- Uneven heating may result in scalding!
- Never heat closed bottles as they may explode!
- Before each use, softly squeeze vent and drinking hole/slit to ensure it is open.
- Cooling down a PP bottle that has been filled with hot liquid (> 80°C) and sealed with the lid afterwards, could cause the bottle to deform permanently.

3.3.2 Cleaning

- Before first use place in boiling water for 5 minutes, use sufficient water to avoid damage.
- To ensure hygiene, clean thoroughly using mild washing up liquid and sterilise before every further use. We recommend to use the NUK Steriliser Vapo Rapid for sterilisation.
- Always dismantle the bottle for boiling or steam sterilising (temperature above

80 °C) into its component parts and leave it open, otherwise deformation may result.

- Certain constituents of drinks (e.g. carrot juice) may discolour the material or affect its odour (e.g. fennel tea). This has no effect on the fitness for use or safety of the bottle.
- Teats and bottles do not belong in the dishwasher – dishwasher detergents can cause material damage!
- For sterilisation in a microwave oven we recommend using the NUK Microwave Steriliser (please follow the instructions of the steriliser/ the device!).

3.3.3 For your child's safety and health: **WARNING!**

- Always use this product with adult supervision.
- Never use feeding teats as a soother.
- Continuous and prolonged sucking of fluids will cause tooth decay. This applies in particular to sweetened drinks and fruit/vegetable juices.
- Always check food temperature before feeding.

- **Always check the teat for signs of excessive wear by pulling the teat in all directions before each use especially when the child has teeth. Replace when sticky from use, visibly damaged or showing signs of tears or bites to avoid the danger of small parts being swallowed.**
- For safety and hygiene purposes, replace the teat after 1-2 months use.
- Do not expose the teat to direct sunlight/ heat or leave in sterilising solution for longer than the stated time as this can damage the teat.
- Store hygienically in a dark and dry place.
- Do not dip in medicine!
- If the bottle is damaged from the inside (e.g. by scratches), it should be replaced for hygiene reasons.

3.3.4 For products which include a seal or protective cap

- Keep all components not in use out of the reach of children.

Note: Use only original NUK Parts. NUK Product complies with EN 14350.

4. Key parts (Figure 0)

- | | | | |
|----|---------------------------|----|-------------------------------------|
| 1 | Silicone cushion | 12 | Teat |
| 2 | Breast shield | 13 | Screw ring |
| 3 | Slot for pump rod | 14 | Protective cap |
| 4 | Pump lever | 15 | Stand for NUK Breast Milk Container |
| 5 | Handle fixture | | |
| 6 | Pump rod | | |
| 7 | Pump diaphragm | | |
| 8 | Catch for handle | | |
| 9 | Lip valve | | |
| 10 | NUK Breast Milk Container | | |
| 11 | Sealing disc | | |

5. Taking the pump apart (Figures A to F)

1. Take the silicone cushion **1** out of the breast shield **2** (Fig. **A**).
2. Holding the end of the handle, pull the pump lever **4** careful and straight upwards, until the handle fixture **5** slips out of the catch for the handle **8** (Fig. **B**).
Attention! Don't pull/turn the pump lever to the side, because the pump rod may break.
3. Carefully remove the pump lever **4** from the pump rod **6** (Fig. **B**).
4. Take the pump diaphragm **7** and pump rod **6** out of the breast shield **2** (Fig. **C**).
5. Ease the pump rod **6** out of the pump diaphragm **7** (Fig. **D**).
6. Unscrew the NUK Breast Milk Container **10** from the breast shield **2** (Fig. **E**).
7. Pull the lip valve **9** out of the breast shield **2** (Fig. **F**).
8. Finally, clean the individual parts as described in "6. Cleaning and sterilising".

6. Cleaning and sterilising

Hygiene is of the utmost priority when handling breast milk and all parts that come into contact with it. **The pump should therefore only be used in a clean and sterile condition.**

Warning! Clean, sterilise and quickly dry all parts (except the stand **15**) thoroughly before using for the first time and immediately **after each use**.

6.1 Cleaning

Warning! Do not use cleaning utensils (such as brushes or scourers) to clean the lip valve **9**. These can cause damage.

1. Clean all parts with warm water and a little washing up liquid (or NUK Rinsing Agent).
 2. Rinse off all parts in running water.
 3. Next, sterilise all parts, without the stand **15** (see "6.2 Sterilising").
- All parts (not the stand **15** and NUK Breast Milk Container **10**) can also be cleaned in the dishwasher.

Note: When cleaning parts in the dishwasher, food residues may cause discolouration of the materials.

6.2 Sterilising

Before sterilising, clean as described in "6.1 Cleaning".

- **Important!** Under no circumstances sterilise the stand **15**, as this would deform the material.
- Use a NUK Steam Steriliser. Follow the operating instructions associated with the device.

Alternatively,

- Boil the parts for 5 minutes in a pan of vigorously boiling water. Ensure there is sufficient water in the pan to avoid damage.
- For sterilisation in a microwave oven we recommend using the NUK Microwave Steriliser (please follow the instructions of the steriliser/ the device!). Follow the manufacturer's instructions when using any sterilisers or chemical solutions in order to prevent damage to the product.

Important! Before using again, allow sterilised parts to cool and dry completely before reassembling the pump.

7. Reassembling the pump (Figures G to N)

1. Insert the pump rod **6** in the pump diaphragm **7**. The lower end of the pump rod **6** must fit in the hole of the pump diaphragm **7**. (Fig. **G**).
2. Position the pump diaphragm **7** in the breast shield **2** and press firmly in place (Fig. **H**).
3. Insert the head of the pump rod **6** in the pump lever **4** (Fig. **I**).
4. Engage the handle fixture **5** in the catch for the handle **8** (Fig. **K**).
5. Insert the lip valve **9** in the breast shield **2** (Fig. **L**).
6. Screw the NUK Breast Milk Container **10** onto the breast shield **2** (Fig. **M**).
7. Position the silicone cushion **1** in the front of the breast shield **2** and pull the edge of the cushion over the breast shield (Fig. **N**).

8. Using the breast pump

Warning! As hygiene is so important, please wash your hands before expressing your milk, and read the information in “10. Hygiene tips” on page 9.

1. Place the funnel of the breast shield over your breast.
 - Make sure that your nipple is in the centre of the silicone cushion **1**. This prevents the nipple becoming sore.
 - Gently press the breast shield **2** against the breast so that the silicone cushion **1** fits closely and no air is sucked in from the side.
 - As far as possible, hold the pump vertically.
2. Press the pump handle **4** then release again slowly.

Note: The further you press the pump lever **4**, the greater the suction. To reduce the suction, do not press the pump lever **4** as far.
3. After expressing, place the breast pump in the stand **15**. The pump must not be allowed to topple over as milk might escape.
4. Unscrew the breast shield **2** from the NUK Breast Milk Container **10**.
5. Firmly seal the NUK Breast Milk Container **10** using the supplied screw ring **13** and sealing disc **11**.
6. Using a freezer pen, write the date and time the milk was expressed on a label.
7. Stick the label on the NUK Breast Milk Container **10**.

Please pay attention to the information in “9. Tips for successful expressing”.

9. Tips for successful expressing

Correct expressing takes practice. You will first need to get used to your new pump.

9.1 Frequency of expressing

How often you need to express depends on the reason for expressing. If you will be apart from your child and/or want to sustain milk production, the natural breastfeeding rhythm should be maintained. You should therefore express 6 to 8 times daily, including once during the night. In all other cases you should express entirely according to your personal wishes and requirements.

9.2 Method

Milk should be expressed from each breast in turn.

The amount of time needed to express milk will vary considerably from one person to another.

9.3 Quantity of milk

Don't be surprised if, at first, it takes a little while for the milk to start flowing properly. This is not unusual. In fact it is quite normal because **milk production adapts itself to daily requirements.**

This means that the quantity of milk produced may vary considerably.

- Always make sure that you have plenty to drink.

9.4 Let-down reflex

During natural breastfeeding, placing the baby on the breast stimulates the let-down reflex and milk begins to flow. You can also stimulate the let-down reflex as follows:

- Find yourself a quiet place to sit, where you can relax fully when using the breast pump.
- Adopt a comfortable and relaxed sitting position.
- Place a warm compress on your breast for a few minutes, to relax the tissue, and massage your breast.
- Ensure you have eye contact with your baby. If this is not possible, look at a photo of your baby.
- The let-down reflex triggers milk flow in both breasts simultaneously. You can use the NUK Breast Milk Collection Shell to collect breast milk leaking from the breast you are not currently expressing.

10. Hygiene tips

Hygiene is especially important when using the breast pump. This applies to all the individual pump parts as well as to your own personal hygiene. We recommend

washing your nipples in clean water daily. Preferably, soap should not be used, as it may irritate the nipples.

11. Storing milk

11.1 NUK Breast Milk Container

If the breast milk is not used immediately,

you must seal the filled breast milk container firmly and store it in the refrigerator or freeze it.

Warning! Do not place the entire NUK JOLIE Breast Pump in the refrigerator, just the breast milk container **10** with screw ring **13** and sealing disc **11**.

Breast milk may be chilled, frozen and defrosted in the breast milk container. The container may also be used for warming

and feeding. It is important that you follow the instructions about handling breast milk which is provided below.

Note: When freezing breast milk, do not fill the container with more than 150 ml (5 fl oz), as breast milk expands as it freezes and the container could burst.

11.2 Important information for handling breast milk

Breast milk	Storage location	Storage periods
Freshly expressed	At room temperature	Max. 6-8 hours
	Refrigerator (not in door)	72 hours at 4 °C
	Refrigerator freezer compartment	2 weeks
	Upright/chest freezer	6 months at -18 °C
Defrosted, unopened	Refrigerator (not in door)	24 hours at 4 °C
Defrosted, opened	Refrigerator (not in door)	12 hours at 4 °C
Defrosting milk	Gradually in refrigerator	Approx. 24 hours at 4 °C
	At room temperature followed by immediate feeding	
	If time is short, defrost container with breast milk in lukewarm water (max. 37°C), then feed immediately.	

- **Warning!** If transporting chilled breast milk make sure that it stays chilled at all times, otherwise the milk could go off and be harmful to your baby. Use a cooler bag with ice packs, for example.
- **Warning!** Breast milk must not be thawed in a microwave as this will destroy valuable nutrients as well as causing possible damage to the container, with a possible risk of explosion. Uneven heating may result in scalding!
- To attain the correct feed temperature of approx. 37 °C, place the container of thawed breast milk in a NUK Baby Food Warmer.

Note: The container must be open to prevent pressure build-up as the milk warms up.

- Shake the milk gently before feeding so that milk and milk fat are properly mixed again.
- Defrosted, warmed or unused milk must be disposed of and not refrozen.
- If you express breast milk more than once within a period of 24 hours, the freshly expressed milk may be added to the previously cooled breast milk.

However freshly expressed milk must be cooled down before being added to milk that is already cool. **The temperatures must be the same (cold on cold).**

- Consult your health visitor, midwife, doctor or pharmacist for detailed information about long-term storage.
- You can find more information about breastfeeding on our NUK website (www.nuk.com).

12. Help with problems

Problem	Reason	Remedy	
Pump has no suction or too little suction	The pump rod 6 is not seated properly in the pump diaphragm 7	Make sure that the pump rod 6 is fully inserted in the hole in the pump diaphragm 7 (see "7. Reassembling the pump (Figures G to N)" on page 8).	
	Pump diaphragm 7 missing, fitted incorrectly or damaged	Insert pump diaphragm 7 correctly (see "7. Reassembling the pump (Figures G to N)" on page 8). If anything is damaged, please contact the NUK Service Center (see "Contact addresses" on page 87).	
	Lip valve 9 is incorrectly positioned	Position lip valve 9 correctly (see "7. Reassembling the pump (Figures G to N)" on page 8).	
	Lip valve 9 missing, fitted incorrectly or damaged	Check that the lip valve 9 is correctly positioned. If anything is damaged, please contact the NUK Service Center (see "Contact addresses" on page 87).	
	Breast milk container 10 is full and lip valve 9 is touching the expressed milk	Change the breast milk container 10.	
	No vacuum is produced		Make sure that the breast shield 2 forms a complete seal around the breast.
			Make sure that the silicone cushion 1 is inserted correctly and is pulled over the edges of the breast shield.
Pump is being held at an angle		Hold the pump as upright as possible.	

13. Technical data

Medical product according to directive 93/42/EEC	Class I
Temperature range	
- Operation	+6 °C to +40 °C
- Storage	0 °C to +40 °C

14. Disposal

If at some stage the appliance is no longer fit for use, the parts can be disposed of along with normal household waste.

15. Warranty

In addition to the seller's warranty obligations arising from the contract to purchase, we as the manufacturers provide a warranty of 24 months from purchase of the appliance as long as the appliance is used properly and the operating instructions are followed. Evidence of the date of purchase and the type of appliance should be provided by means of a receipt.

The purchaser's statutory rights are unaffected by this manufacturer warranty.

During the warranty period, we undertake to remedy any defects attributable to poor materials or workmanship. Worn parts are excluded from the warranty.

Minor deviations from specified quality which do not affect the value or usability of the appliance are not covered by the warranty. We are likewise unable to accept the warranty if the defects on the appliance are due to transport damage for which we are not responsible, if they are attributable to misuse or lack of care or if the appliance has been interfered with by persons not authorized by us to do so.

The warranty service will involve repair, replacement of parts or replacement of the appliance at our discretion. The warranty period will be neither extended nor restarted by the performance of warranty services. The warranty for any spare parts fitted will expire with the warranty for the appliance as a whole.

Extended or different claims, especially those relating to the making good of damage sustained outside the appliance,

are excluded unless liability is compulsory in law.

We will not accept the costs or risks of transport.

An appliance sent in without evidence of the purchase date will be treated as a repair. The appliance will only be repaired following discussion with the customer.

Retain address, article number and LOT No. for possible future reference. The LOT No. can be found on the breast shield (see Fig. O)

Service Centres

Addresses for service can be found in "Contact addresses" on last page.

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NUK is a registered trademark of MAPA GmbH, Germany
MAPA GmbH