D NUK COOL & WARM ERGONOMISCHE BRUST-GELKOMPRESSE

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MAPA GmbH Industriestraße 21-25 D-27404 Zeven

GB NUK COOL & WARM ERGONOMIC BREAST GEL COMPRESS

NUK has created a breast gel compress, which allows you to care for your breasts in a pleasant way, by either warming or cooling them. The ergonomic round shape, which has a space for the nipple and an extended section, which goes under the arm at the side of the breast, guarantees that all mammary and lymph glands are supplied with either a warming or cooling sensation as desired.

Areas of application

- The NUK breast gel compress can be applied either cold or warm and before, during and after breastfeeding. During pregnancy: Even in the first few weeks of pregnancy you will notice changes in your breasts...

- While breastfeeding: Placed on the breast as a warm compress before feeding, the NUK breast gel compress helps promote milk flow, thus offering greater comfort while breastfeeding...

- After weaning: Once you have weaned your infant you can continue to use the NUK breast gel compress as a warm or cold compress on your breast - just as you wish.

Application

Heating the NUK breast gel compress

- 1. Immerse the NUK breast gel compress for 8 minutes in warm water (50°C - not boiling). 2. Remove and test the temperature on the back of your hand.

Cooling the NUK breast gel compress: Place the NUK breast gel compress in the refrigerator for 15 to 45 minutes, depending on how cold you wish it to be.

- 1. Place the NUK breast gel compress in the refrigerator for 15 to 45 minutes, depending on how cold you wish it to be. 2. Now place the cold NUK breast gel compress on the breast (without bra) and experience the calming effect.

Duration of application: The cold compress should be applied for max. 20 minutes, the warm compress max. 8 minutes.

Cleaning: Before using for the first time you should wash the compress in warm water (max. 50°C).

- 1. Before using for the first time you should wash the compress in warm water (max. 50°C). 2. Each time before using you should check the NUK breast gel compress for traces of dirt, remove and wash in warm water if necessary.

Safety Instructions

- Use correctly the NUK breast gel compress offers the greatest possible security. Do not place the NUK breast gel compress in the freezer or freezing compartment of the fridge to avoid freezer burn.

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- While breastfeeding: Placed on the breast as a warm compress before feeding, the NUK breast gel compress helps promote milk flow, thus offering greater comfort while breastfeeding.

- After weaning: Once you have weaned your infant you can continue to use the NUK breast gel compress as a warm or cold compress on your breast - just as you wish.

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NUK is a registered trademark owned by MAPA GmbH/Germany